

# Product Security Bulletin

IMPORTANT INFORMATION – PLEASE READ AND KEEP

## SentrySuite: Windows 10 Security Update Compatibility

---

Date: 07-OCT-2021 (last update)

### Background

Microsoft is identifying Operating System (OS) and native Microsoft application level vulnerabilities which impact the Windows 10 OS which is part of the environment in which Vyaire Medical's SentrySuite software runs.

While these vulnerabilities do not impact SentrySuite software directly, changes made by Microsoft to remediate these issues may cause compatibility issues with SentrySuite.

#### How Microsoft Is Responding

Microsoft regularly publishes new builds for its Windows 10 OS, called Quality Updates. The Quality Updates may contain important security patches for vulnerabilities in the Windows 10 OS. An overview of Windows 10 Quality Updates may be found here: [Windows 10 update history](#).

#### How Vyaire Is Responding

Vyaire's SentrySuite software has been validated for use with Microsoft Windows 10 version **20H1 (2004)** and **20H2**.

As Microsoft releases new Quality Updates for Windows 10 20H1 and 20H2, each version of SentrySuite compatible with Windows 10 which is not considered End-Of-Life is tested for compatibility with those new updates.

Each Quality Update contains the cumulative changes of all previously released updates. Testing of Quality Updates which are not security related maybe delayed.

This document will be regularly updated to reflect the outcome of Vyaire's compatibility tests and therefore provide a guideline which patches may be installed without compromising SentrySuite operation.

# Product Security Bulletin

IMPORTANT INFORMATION – PLEASE READ AND KEEP

## Compatibility test results

Release date	Windows 10 build	Relevant Microsoft KB	Security updates in build	SentrySuite compatibility	Notes
10-AUG-2021	<b>20H2:</b> 19042.1165 <b>20H1:</b> 19041.1165	<a href="#">KB5005033</a>	Yes	<b>3.20.2</b> ✓ <b>3.10.6</b> ✓ <b>3.0.9</b> ✓	
13-JUL-2021	<b>20H2:</b> 19042.1110 <b>20H1:</b> 19041.1110	<a href="#">KB5004237</a>	Yes	<b>3.20.1</b> ✓ <b>3.10.5</b> ✓ <b>3.0.8</b> ✓	
06-JUL-2021 (out of band)	<b>20H2:</b> 19042.1083 <b>20H1:</b> 19041.1083	<a href="#">KB5004945</a>	Yes	<b>3.20.1</b> ✓ <b>3.10.5</b> ✓ <b>3.0.8</b> ✓	
08-JUN-2021	<b>20H2:</b> 19042.1052 <b>20H1:</b> 19041.1052	<a href="#">KB5003637</a>	Yes	<b>3.20.1</b> ✓ <b>3.10.5</b> ✓ <b>3.0.8</b> ✓	
11-MAY-2021	<b>20H2:</b> 19042.985 <b>20H1:</b> 19041.985	<a href="#">KB5003173</a>	Yes	<b>3.20.1</b> ✓ <b>3.10.5</b> ✓ <b>3.0.8</b> ✓	
13-APR-2021	<b>20H2:</b> 19042.928 <b>20H1:</b> 19041.928 <b>1909:</b> 18363.1500	<a href="#">KB5001330</a> <a href="#">KB5001330</a> <a href="#">KB5001337</a>	Yes	<b>3.20.1</b> ✓ <b>3.10.4</b> ✓ <b>3.0.7</b> ✓	See (4)
09-MAR-2021	<b>20H1:</b> 19041.867 <b>1909:</b> 18363.1440	<a href="#">KB5000802</a> <a href="#">KB5000808</a>	Yes	<b>3.20</b> ✓ <b>3.10.4</b> ✓ <b>3.0.7</b> ✓	
09-FEB-2021 11-FEB-2021	<b>20H1:</b> 19041.804 <b>1909:</b> 18363.1379	<a href="#">KB4601319</a> <a href="#">KB5001028</a>	Yes	<b>3.20</b> ✓ <b>3.10.3</b> ✓ <b>3.0.6</b> ✓ <b>2.21.6</b> ✓	
08-DEC-2020	<b>1909:</b> 18363.1256 <b>1903:</b> 18362.125	<a href="#">KB4592449</a>	Yes	<b>3.10.3</b> ✓ <b>3.0.6</b> ✓ <b>2.21.6</b> ✓	See (3)

# Product Security Bulletin

IMPORTANT INFORMATION – PLEASE READ AND KEEP

Release date	Windows 10 build	Relevant Microsoft KB	Security updates in build	SentrySuite compatibility	Notes
13-OCT-2020	<b>1909:</b> 18363.1139 <b>1903:</b> 18362.1139	<a href="#">KB4577671</a>	Yes	<b>3.10.3</b> ✓ <b>3.0.6</b> ✓ <b>2.21.6</b> ✓	
16-SEP-2020	<b>1909:</b> 18363.1110 <b>1903:</b> 18362.1110	<a href="#">KB4577062</a>	No	<b>3.10.3</b> ✓ <b>3.0.6</b> ✓ <b>2.21.6</b> ✓	
08-SEP-2020	<b>1909:</b> 18363.1082 <b>1903:</b> 18362.1082	<a href="#">KB4574727</a>	Yes	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	
20-AUG-2020	<b>1909:</b> 18363.1049 <b>1903:</b> 18362.1049	<a href="#">KB4566116</a>	No	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	
11-AUG-2020	<b>1909:</b> 18363.1016 <b>1903:</b> 18362.1016	<a href="#">KB4565351</a>	Yes	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	
21-JUL-2020	<b>1909:</b> 18363.997 <b>1903:</b> 18362.997	<a href="#">KB4559004</a>	No	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	
14-JUL-2020	<b>1909:</b> 18363.959 <b>1903:</b> 18362.959	<a href="#">KB4565483</a>	Yes	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	
16-JUN-2020	<b>1909:</b> 18363.904 <b>1903:</b> 18362.904	<a href="#">KB4567512</a>	No	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	
09-JUN-2020	<b>1909:</b> 18363.900 <b>1903:</b> 18362.900	<a href="#">KB4560960</a>	Yes, see <a href="#">Microsoft Security Update Guide</a>	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	See (2)
12-MAY-2020	<b>1909:</b> 18363.836 <b>1903:</b> 18362.836	<a href="#">KB4556799</a>	Yes, see <a href="#">Microsoft Security Update Guide</a>	<b>3.10.2</b> ✓ <b>3.0.5</b> ✓ <b>2.21.6</b> ✓	

# Product Security Bulletin

IMPORTANT INFORMATION – PLEASE READ AND KEEP

Release date	Windows 10 build	Relevant Microsoft KB	Security updates in build	SentrySuite compatibility	Notes
21-APR-2020	1903:18362.815	<a href="#">KB4550945</a>	No	3.10.2 ✓ 3.0.5 ✓ 2.21.6 ✓	
14-APR-2020	1903:18362.778	<a href="#">KB4549951</a>	Yes, see <a href="#">Microsoft Security Update Guide</a>	3.10.2 ✓ 3.0.5 ✓ 2.21.5 ✓	
30-MAR-2020	1903:18362.753	<a href="#">KB4554364</a>	No	3.10.1 ✓ 3.0.4 ✓ 2.21.5 ✓	
24-MAR-2020	1903:18362.752	<a href="#">KB4551762</a>	No	3.10.1 ✓ 3.0.4 ✓ 2.21.5 ✓	
12-MAR-2020	1903:18362.720	<a href="#">KB4551762</a>	Yes, see <a href="#">Microsoft Security Update Guide</a>	3.10.1 ✓ 3.0.4 ✓ 2.21.5 ✓	See (1)
10-MAR-2020	1903:18362.719	<a href="#">KB4540673</a>	Yes, see <a href="#">Microsoft Security Update Guide</a>	3.10.1 ✓ 3.0.4 ✓ 2.21.5 ✓	
27-FEB-2020	1903:18362.693	<a href="#">KB4535996</a>	No	3.10.1 ✓ 3.0.4 ✓ 2.21.5 ✓	
11-FEB-2020	1903:18362.657	<a href="#">KB4532693</a>	Yes, see <a href="#">Microsoft Security Update Guide</a>	3.10.1 ✓ 3.0.4 ✓ 2.21.5 ✓	
28-JAN-2020	1903:18362.628	<a href="#">KB4532695</a>	No	3.10.1 ✓ 3.0.4 ✓ 2.21.5 ✓	
14-JAN-2020	1903:18362.592	<a href="#">KB4528760</a>	Yes, see <a href="#">Microsoft Security Update Guide</a>	3.10.1 ✓ 3.0.4 ✓ 2.21.5 ✓	

# Product Security Bulletin

IMPORTANT INFORMATION – PLEASE READ AND KEEP

Vyaire recommends installing any relevant security updates that have been tested as compatible per the compatibility test results table above.

## Compatibility test notes

- (1) Microsoft OneNote update is installed with this build and sets the default printer to OneNote. Default printer needs to be manually changed back to previously selected default printer. This does not affect SentrySuite.
- (2) Microsoft introduced an issue in this build which may cause issues with printing to a physical or virtual printer. Vyaire does not recommend installing this build but instead to install build 904, released by Microsoft on 16-JUN-2020 which mitigates this issue.
- (3) This will be the last Windows update released by Microsoft for Windows 10 1903. Windows 10 1903 has reached “End of Life” (EOL) status as of 08-DEC-2020
- (4) This will be the last Windows update released by Microsoft for Windows 10 1909. Windows 10 1909 has reached “End of Life” (EOL) status as of 11-MAY-2021.

# Product Security Bulletin

IMPORTANT INFORMATION – PLEASE READ AND KEEP

## SentrySuite and Windows 10 compatibility testing plan

All Windows 10 versions released by Microsoft have a defined “End of Life” (EOL) date after which Microsoft won’t release any more quality or security updates for the version. For details and EOL dates, please see Microsoft’s [Windows 10 release information website](#) and the [Windows Lifecycle FAQ](#).

The following table explains which SentrySuite version is tested against which Windows 10 feature version:

Windows 10 Version	Windows 10 EOL date	Compatibility test status	SentrySuite Version			
			3.20	3.10	3.0	2.21
<b>1803</b>	<b>12-NOV-2019</b>	<b>Tests stopped</b>	--	--	x	x
<b>1809</b>	<b>10-NOV-2020</b>	<b>Tests stopped</b>	--	--	x	x
<b>1903 (19 H1)</b>	<b>08-DEC-2020</b>	<b>Tests stopped 08-DEC-2020</b>	--	x	x	x
<b>1909 (19 H2)</b>	<b>11-MAY-2021</b>	<b>Tests stopped 13-APR-2021</b>	x <sup>1</sup>	x	x	x <sup>2</sup>
<b>2004 (20 H1)</b>	<b>14-DEC-2021</b>	<b>Tests ongoing</b>	x	x	x	x <sup>2</sup>
<b>20 H2</b>	<b>10-MAY-2022</b>	<b>Tests ongoing</b>	x	x	x	
<b>21 H1</b>	<b>13-DEC-2022</b>	<b>On roadmap</b>	x	x	x	
<b>21 H2</b>	<b>not yet released</b>	<b>On roadmap</b>	x	x		

<sup>1</sup>Tests for Windows 10 1909 and SentrySuite 3.20 will commence once SentrySuite 3.20 has been released by Vyairé.

<sup>2</sup>Tests for SentrySuite 2.21 will be stopped after end of March 2021.

Vyairé does not recommend using Windows 10 versions which have been marked as EOL by Microsoft.

These Windows 10 versions will not receive regular quality and security updates by Microsoft anymore any may be vulnerable to threats that emerged after the EOL date.

# Product Security Bulletin

IMPORTANT INFORMATION – PLEASE READ AND KEEP

## Generic controls

- Ensure your data has been backed up and stored according to your individual process and that your disaster recovery procedures are in place.
- Update your anti-virus and malware protection, where available.

For product or site-specific concerns, contact your Vyair service representative.

For more information on Vyair's proactive approach to product security and vulnerability management, contact us at [productsecurity@vyaire.com](mailto:productsecurity@vyaire.com) or visit [www.vyaire.com/product-security](http://www.vyaire.com/product-security)




**Timo Kosig**  
timo.kosig@vyaire.com  
Product Security Leader  
Vyair Medical GmbH



**Holger Schmitt**  
holger.schmitt@vyaire.com  
Senior Product Manager Software and IT integration  
Vyair Medical GmbH

### GLOBAL HEADQUARTERS

Vyair Medical, Inc.  
26125 North Riverwoods Blvd  
Mettawa, IL 60045  
USA

 Vyair Medical GmbH  
Leibnizstrasse 7  
97204 Hoechberg  
Germany

### AUSTRALIAN SPONSOR

Vyair Medical Pty Ltd  
Suite 5.03, Building C  
11 Talavera Road  
Macquarie Park, NSW, 2113  
Australia

**CE** 0123